

Sharpness in your handle^{le}




BAYHA[®] Scalpels
with changeable blades

Advantages:


- All blade types fit in all types of BAYHA - handles
- Approved locking system – guarantees a solid grip of the blade in the handle
- Lateral entering of the blade into the handle
- Safe in handling

BAYHA[®] Blades

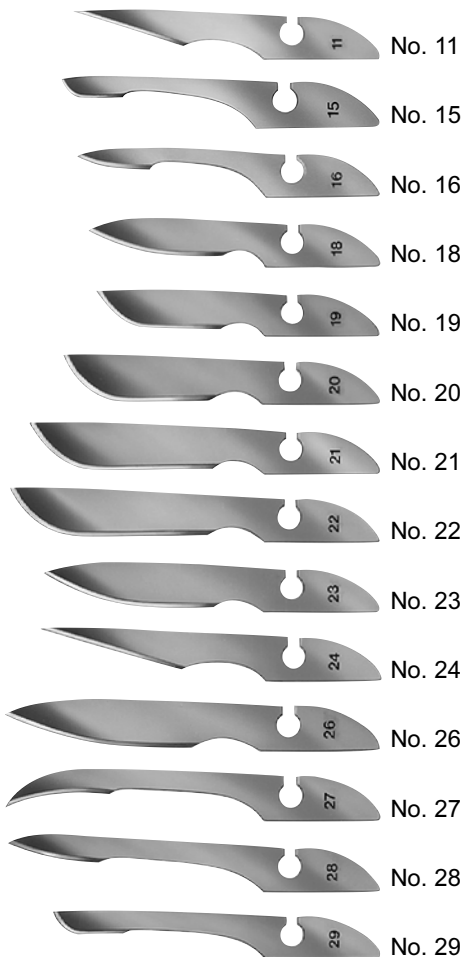

Blades in carbon steel, hardened.
1 sales unit = 144 pieces (12 dozen)
Individually packed:
Sterile blades in Aluminium-Foil.
Non-sterile blades in Paper-Foil.

Perfect – the approved BAYHA-locking system

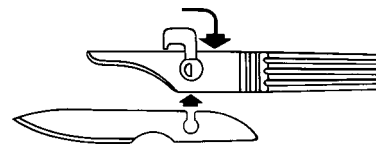

1. Pull the bolt upwards, the blade is loose.
2. Put in the blade as indicated above.
3. Press the bolt in the direction of the arrow, then the blade is fixed.

BAYHA[®] Handles

Stainless steel,
suitable for all blade forms.



Fig. 1
13 cm



Fig. 2
13 cm



Fig. 3
20 cm



Fig. 4
16 cm



Fig. 5
15 cm